

Nelli® Frequently Asked Questions for Patient

How does it work/what is it all about?

The system recognizes seizures (and other abnormal movements) and highlights them for the doctor for further investigation.

Where is the information stored?

All information is stored locally. The sections containing epileptic events and seizures are sent to the cloud service to be analyzed by the AI. All information is highly secured when stored and transferred.

Can somebody hack the system?

We take security very seriously. All measures are taken for this not to happen.

Who will see the video?

Only assigned experts of Neuro Event Labs are allowed to view the recordings in order to review and validate the findings to increase the accuracy of the report. The report is delivered to the doctor and only includes the seizures or other medical findings.

Is the video camera on all the time?

Yes, but it will store the recording only when the AI recognizes or is suspecting there is an epileptic symptom or seizure.

Should I try to go to bed if I feel seizure coming during day?

That would be very good, but the assumption is that Nelli captures the seizures that happen when resting on the bed according to your usual rhythm. The more information of the seizures collected, the better.

Does it make a sound?

It may make low whirl sound, like a laptop computer.

Can I leave the room for e.g. the toilet during recording?

Yes you can. You can stay in other parts of the room during the day. Also you may leave the bed while resting for a short moment when you need to. No findings will be captured during your absence from the bed, but it won't disturb the recording or analyses either.

Does it need to be connected to the internet?

The system is connected automatically via a built-in modem. That's what the antennas are for.